

## Lava™ Ceram



### COMBINATION TABLE

VITA-Classic-Colors	A1	A2	A3	A3,5	A4	B1	B2	B3	B4	C1	C2	C3	C4	D2	D3	D4	BLEACH COLORS*		
																	W1	W2	W3
<b>Shoulder Porcelains</b>	SH 1	SH 3		-	-	SH 1	-	SH 5		-	-	-	-	-	-	-	-	-	-
<b>Frame Work Modifier</b>	MO A1	MO A2	MO A3	MO A3,5	MO A4	MO B1	MO B2	MO B3	MO B4	MO C1	MO C2	MO C3	MO C4	MO D2	MO D3	MO D4	MO W1	MO W2	MO W3
<b>Dentin Porcelains</b>	D A1	D A2	D A3	D A3,5	D A4	D B1	D B2	D B3	D B4	D C1	D C2	D C3	D C4	D D2	D D3	D D4	D W1	D W2	D W3
<b>Enamel Porcelains</b>	E 2	E 2	E 3	E 3	E 4	E 1	E 1	E 3	E 3	E 4	E 3	E 3	E 4	E 4	E 3	E 3	E 2	E 2	E 2

\*3M ESPE

### INDIVIDUAL SHADES

Enamel Effect	
Polar	E 5
Sun	E 6
Transparent Opal Porcelains	
Neutral	T 1
Yellow	T 2
Blue	T 3
Grey	T 4
Clear	CL

Fluorescent Transparent Opal	
Diamant	ICE1
Flamingo	ICE2
Transparent Nature	
Pearl	N1
Frost	N2
Crystal	N3
Snow	N4
Cervical Incisal Transpa	
Lychee	CI 1
Apricot	CI 2
Lemon	CI 3

Opaque Dentin	
Ivory	OD 1
Curry	OD 2
Amber	OD 3
Siena	OD 4
Flu- In Dentin	
Cream	FL 1
Gold	FL 2
Corn	FL 3
Mango	FL 4
Nougat	FL 5
Olive	FL 6

Magic Intensive	
Ocean blue	I 1
Atlantis	I 2
Maroon	I 3
Havanna	I 4
Orange	I 5
Khaki	I 6
Vanilla	I 7
Honey	I 8
Gingiva	I 9
Violet	I 10

Stains	
Ocean blue	S 1
Atlantis	S 2
Maroon	S 3
Havanna	S 4
Orange	S 5
Khaki	S 6
Vanilla	S 7
Honey	S 8
Gingiva	S 9
Violet	S 10

Gingiva	
Light	G 1
Dark	G 2
Correction Porcelain	
Neutral	CR 1
Yellow	CR 2
Glaze	G



SHORT VENEERING INSTRUCTION

## Natural Esthetics

*Layering scheme\**



Coping with layer of high-fluorescent Lava™ coping modifier.




Cut or scrape back to the size of the dentine core. The mamelon tips will be at about the same level as the adjacent teeth.



Extend using two incisal materials (Enamel and Enamel + Clear).



Apply mamelon materials consisting of framework modifiers and Stain colours or Magic Intensive Shades in a thin layer.

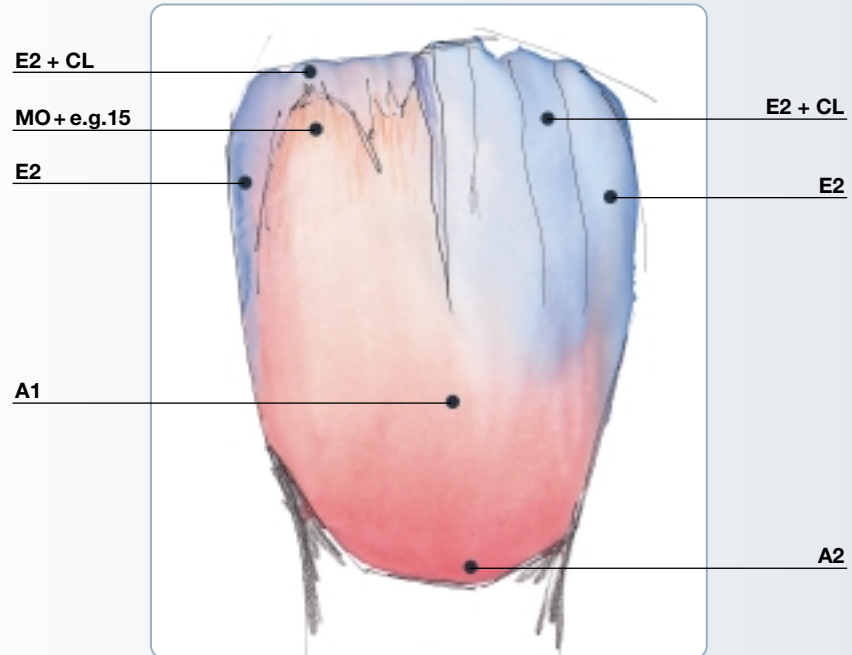

Complement the shape using alternating layers of incisal materials.



Completed crown.